



University of Missouri Canine Genetics Laboratory

Blood Sample Submission for Rottweiler Epilepsy Research

From each dog have 4 to 5 ml of blood collected into an EDTA tube (lavender-top).

Label each tube with the dog's name and owner's last name.

Complete a copy of the Epilepsy Questionnaire for each dog.

If you are not going to ship the blood right away, store refrigerated until ready to ship.

Pack the samples with the completed questionnaires and ship to us (overnight shipping preferred).
(If the weather is hot, please package the sample with a cold pack. Otherwise, a cold pack is not necessary).

Ship the package to:

Canine DNA Testing/Rottweiler Epilepsy
320 Connaway Hall/1500 Bouchelle Ave.
University of Missouri
Columbia, MO 65211 USA

For questions, contact us at: mucvmk9genetics@missouri.edu

Phone: 573-884-3712